TIP-GUARD™

Instrument Protectors

Scanlan Group B.V.
Aalsmeerderweg 610 • 1437 EJ Rozenburg-Schiphol
The Netherlands

SCANLAN ★

®

INTERNATIONAL

Surgical products designed and manufactured by the Scanlan family since 1921

SCANLAN INTERNATIONAL, INC.
ONE SCANLAN PLAZA
SAINT PAUL, MINNESOTA 55107 • U.S.A.
800-328-9458 • 651-298-0997 • FAX 651-298-0018

www.scanlaninternational.com

© 2021 Scanlan International, Inc.

MK-IFU-TG Rev. D

TIP-GUARD™ Instrument Protectors

Clinical Benefits:

TIP-GUARD™ instrument protectors are not intended for direct patient contact or use. TIP-GUARD™ instrument protectors are surgical instrument tip protectors used to guard against damage or injury to tips and edges of surgical instruments and their users. TIP-GUARD™ instrument protectors are available in nine (9) different sizes in colored, tinted, clear, or vented form. TIP-GUARD™ instrument protectors are made of soft and pliable radiopaque PVC that will fit securely over the tips of surgical instruments. TIP-GUARD™ instrument protectors are supplied non-sterile and may be steam or flash autoclaved and are ethylene oxide (EO) sterilizable.

Intended User:

The intended users of the TIP-GUARD™ instrument protectors are Sterile Processing Department Personnel and associated areas in the health care facility setting.

Intended Use:

SCANLAN® TIP-GUARD™ instrument protectors are intended to guard against damage to tips and edges of surgical instruments.

Instructions for Use:

Choose the TIP-GUARD™ instrument protector style that best protects the instrument. Place the TIP-GUARD™ instrument protector over the tip of a clean surgical instrument. Instruments may be packaged in sterile packaging systems (e.g., pouch, wrapped tray, closed container) for sterilization (steam or EO). Dispose of all TIP-GUARD™ instrument protectors following initial use.

NOTICE: Clear and tinted TIP-GUARD™ instrument protectors may cloud when autoclaved. This in no way affects the sterility or performance of the product.

- **Contraindications:**
- TIP-GUARD™ instrument protectors are not intended to be reprocessed as they are a single use product.

TIP-GUARD™ instrument protectors are not intended for implantation.

TIP-GUARD™ instrument protectors are not intended for use during decontamination/cleaning.

- **Single-Use Risk of Reuse Statement:**
- Reprocessing or re-use of this SINGLE-USE DEVICE is prohibited. Any such reuse or reprocessing of this SINGLE-USE DEVICE shall void any warranty or liability by the manufacturer for injury or loss associated with inappropriate use. Reuse of single-use devices creates a potential risk of patient or user infections. Contamination of the device may lead to injury, illness, or death of the patient.

- **Requirements for Special Facilities, Training, or Particular Qualifications of the User and/or Other Persons:**
- Not Applicable

- **Methods for Safe Disposal:**
- The user is responsible to dispose of the device and packaging according to local and national requirements.

- **Incident Reporting Notice:**
- Any serious incident that has occurred in relation to the device should be reported to Scanlan International, Inc. and the competent authority of the Member State in which the user and/or patient is established.

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