Traveler™

Autoclavable Pouch

EC REP Scanlan Group B.V.
Aalsmeerderweg 610 • 1437 EJ Rozenburg-Schiphol
The Netherlands

SCANLAN ★

INTERNATIONAL

Surgical products designed and manufactured by the Scanlan family since 1921

SCANLAN INTERNATIONAL, INC.
ONE SCANLAN PLAZA
SAINT PAUL, MINNESOTA 55107 • U.S.A.
800-328-9458 • 651-298-0997 • FAX 651-298-0018 • www.scanlaninternational.com

© 2021 Scanlan International, Inc.

MK-IFU-TRAV Rev D.

Traveler™ Autoclavable Pouch

Clinical Benefits:

SCANLAN® TRAVELER™ Autoclavable Pouch is a single-use device providing users with a means of identifying, or providing information regarding, the contents of sterilization trays or other items that may enclose instruments/devices for means of sterilization or storage. They can be used in steam sterilization. The pouch is not intended to sterilize or maintain sterility of items placed inside it.

Intended User:

The intended users of the TRAVELER™ Autoclavable Pouch are Sterile Processing Department Personnel and associated areas in the health care facility setting.

Intended Use:

The TRAVELER™ Autoclavable Pouch is intended for use on the outside of rigid sterilization containers to house instrument count sheets or other forms of necessary communication during an autoclave sterilization cycle.

Instructions for Use:

1. Fold the instrument countsheet(s) as necessary.
2. Open the autoclavable pouch and place folded count sheet(s) inside.
3. Insert the enclosed fastener (if included) through both holes to close the pouch and secure count sheet.
4. Place the fastener (if included) around the handle of tray or container and close fastener. Ensure fastener is locked so pouch stays in place during use.
5. After sterilization cycle is complete and container is removed from sterilizer, verify the autoclavable pouch remains attached to the rigid container.
6. To remove pouch, detach from the tray or container by pressing the release button on the fastener and removing it from the container.

⚠ Contraindications:

TRAVELER™ is not intended to be reprocessed as it is a single use product.

TRAVELER™ is not intended for use during decontamination/cleaning.

TRAVELER™ is not intended for use with sterilization modalities other than steam.

TRAVELER™ is not a sterilization container and is not intended to hold sterile items within it.

⚠ Single-Use Risk of Reuse Statement:

Reprocessing or re-use of this SINGLE-USE DEVICE is prohibited. Any such reuse or reprocessing of this SINGLE-USE DEVICE shall void any warranty or liability by the manufacturer for injury or loss associated with inappropriate use. Reuse of single-use devices creates a potential risk of patient or user infections. Contamination of the device may lead to injury, illness, or death of the patient.

⚠ Requirements for Special Facilities, Training, or Particular Qualifications of the User and/or Other Persons:

Not Applicable

⚠ Methods for Safe Disposal:

The user is responsible to dispose of the device and packaging according to local and national requirements.

⚠ Incident Reporting Notice:

Any serious incident that has occurred in relation to the device should be reported to Scanlan International, Inc. and the competent authority of the Member State in which the user and/or patient is established.

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